Nursing Bench 1000 Owners Manual

Nursing bench 1000 – Operation and maintenance manual

en

English	3
Dansk	21
Deutsch	38
Français	55
Nederlands	72
Svenska	89
Norsk	106
Español	123
中文	140

- en** The guarantee covers faults or defects in material or manufacture within a period of 3 years. Products are subject to minor technical modifications and design deviations. E. & O.E.
- dk** Garantien dækker i 3 år fra fakturadato for materiale- og fabrikationsfejl. Der tages forbehold for trykfejl, tekniske ændringer og modelafvigelser.
- de** Die Garantie gilt für drei Jahre ab Rechnungsdatum für Material- und Herstellungsfehler. Druckfehler, technische Änderungen und Modellabweichungen vorbehalten.
- fr** La garantie couvre les vices de fabrication et de matériaux pour une durée de 3 ans à partir de la date de facturation. Sous réserve de modifications techniques et d'erreurs d'impression.
- nl** De garantie is tot 3 jaar na factuurdatum geldig voor materiaal- en fabricagefouten. Onder voorbehoud van drukfouten, technische wijzigingen en modelafwijkingen.
- sv** Garantin gäller i 3 år från fakturadatum för material- och fabrikationsfel. Med reservation för tryckfel, tekniska ändringar och modellavvikelser.
- no** Garantien gjelder i 3 år fra fakturadato for material- og fabriksjonsfeil. Med forbehold for trykfeil, tekniske endringer og modellforskjeller.
- es** La garantía cubre fallos de material o fabricación durante los tres años posteriores a partir de la fecha de factura. Mantenemos reservas en caso de problemas de presión, modificaciones técnicas y variaciones de modelo.
- 中文** 保修涵盖 3 年内出现的材料或制造故障或缺陷。产品的技术和设计可能稍微发生变化，恕不另行通知。

Content

Symbols used in this manual	4
Specific safety	4
General safety	5
Product label	6
Intended use	8
Intended operator profile	8
Intended user profile	8
Operating environment	8
Assembly and installation	9
Versions	10
Operating instructions	11
Height adjustment	12
Cleaning	13
Disinfection	13
Maintenance	14
Service inspection	14
Trouble-shooting	16
Technical data	17
Dimensions (mm)	18
Dimensions (inches)	19
Disposal and recycling	20

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Electricity or potentially dangerous voltage.

Head injury warning.

Specific safety

- Never leave a person lying on the bench without supervision.
- Always be careful when moving a person to and from the bench.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.


CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label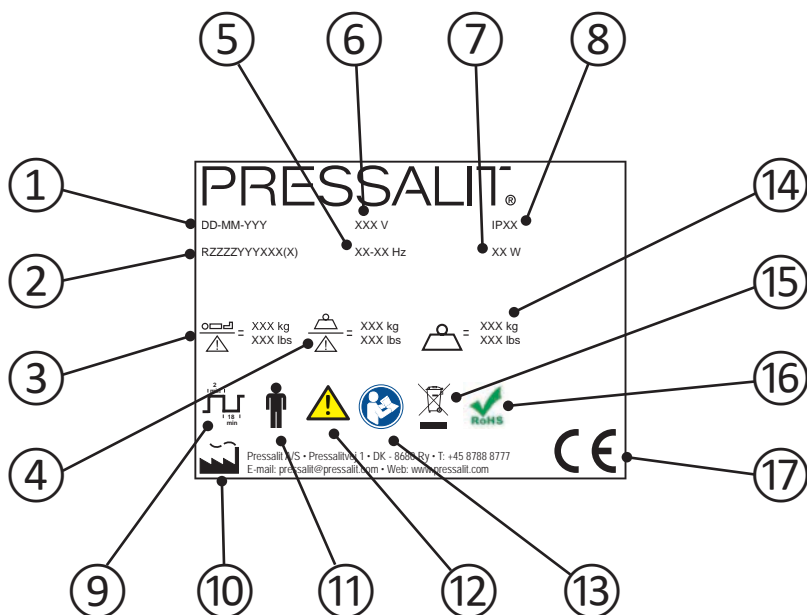

en

① Production date

② Article number

③ Maximum user weight


④ Safe working load

⑤ Frequency

⑥ Mains voltage

-
- ⑦ Maximum power consumption
-
- ⑧ IP class
-
- ⑨ Duty cycle The motor on height-adjustable products can be used for a maximum of 2 minutes of continuous operation, followed by 18 minutes pause.
-
- ⑩ Name and address of manufacturer
-
- ⑪ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
-
- ⑫ Consult the manual for use for important cautionary information such as warnings and precautions.
-
- ⑬ Consult manual for use
-
- ⑭ Product weight
-
- ⑮ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
-
- ⑯ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
-
- ⑰ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and with Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at www.pressalitcare.com.

Product label placement ①.

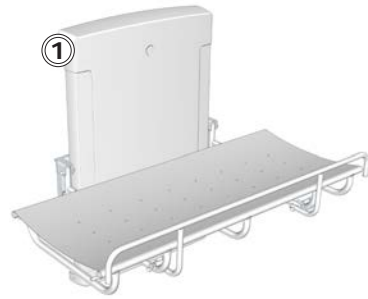Intended use

This product is intended for bathing a person who is unable to stand up or sit down. The person is lying horizontally on the bench.

Intended operator profile

The product is to be used by personnel trained in health care and trained in washing a person in their care.

Intended user profile

The product can be used by all - adults, children, young, old, whose weight does not exceed the **maximum load of 150 kg / 330 lbs.**

en Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in a stand-alone environment, in private homes, nursing homes and hospitals under normal working conditions.

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

Assembly and installation

For assembly and installation, please refer to the assembly instruction supplied with the product.

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

Pressalit Care
 Pressalitweg 1
 DK 8460 Tj
 Denmark

Versions

The nursing bench comes in four versions.


R8400 and R8402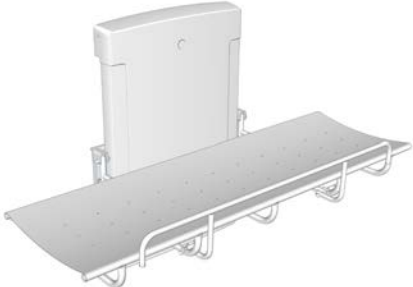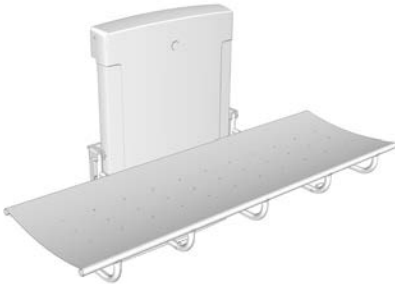

R8401 and R8403

en

Operating instructions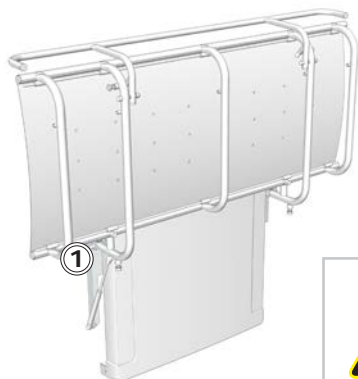

Support the bench using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the bench up against the wall after use.



Take care to avoid injury to hands when in the vicinity of closing mechanical parts ①.

To pull the (optional) safety rail up: Swing the rail upwards from under the bench. The safety rail will automatically lock into position.


en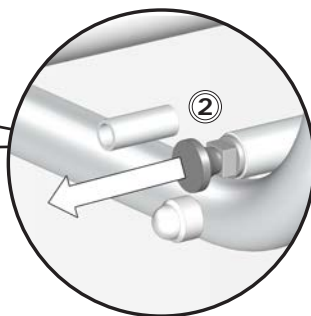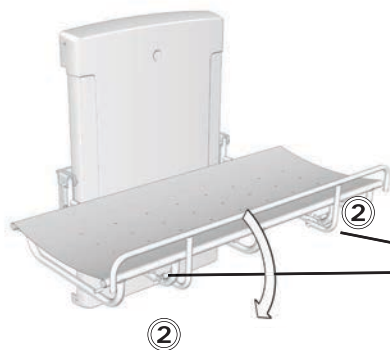

To fold the safety rail away: Release the two locks ② simultaneously and swing the safety rail down under the bench.

Height adjustment

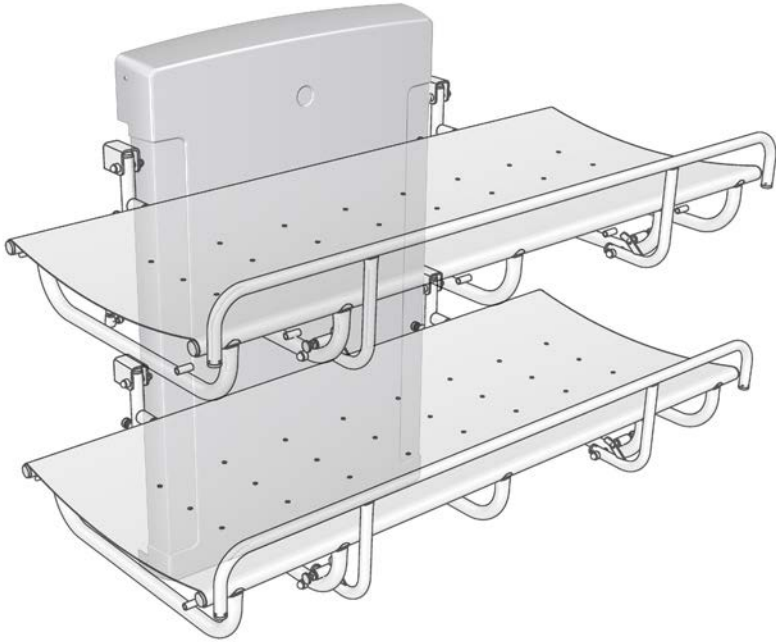When the bench is installed at the recommended height, it can be adjusted between 480 and 880 mm (18.9" and 34.6") above the floor.

The height adjustment is controlled by the wired remote control unit.

Press and hold the Up button to raise the bench, and press and hold the Down button to lower the bench.

Make sure that there is no objects under the nursing bench when operating the height adjustment system.

Cleaning

Clean the nursing bench after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the nursing bench directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chloride-based cleaning or disinfecting products.

Disinfection

When using the nursing bench for multiple patients, it is necessary to both clean and disinfect the nursing bench between uses on different patients to prevent the risk of cross infection.

A list of suitable cleaning agents and disinfectants can be found at www.pressaliticare.com

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

Maintenance

Expected service life of the nursing bench is 10 years, provided that maintenance is carried out as described below:


It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the bench to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that service inspection is done once a year.


Service inspection

- Check the hinges ① on the bench and the safety rail ② . Tighten if necessary.
- Check the PVC canvas ③ for damage and replace if necessary.

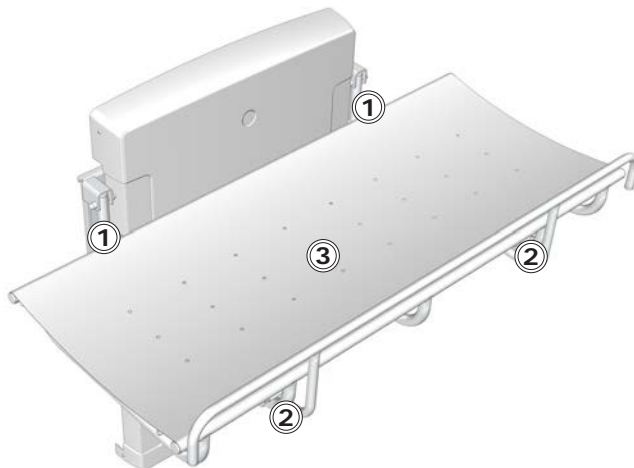- Remove the upper and lower retaining screws ④ on the cover and remove it carefully.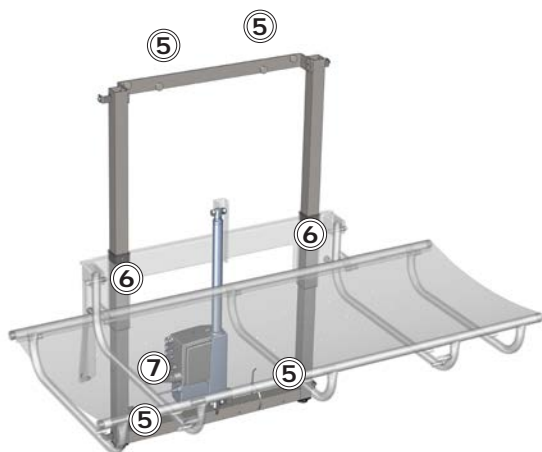

- Check the wall brackets ⑤ and tighten if necessary.
- Oil moving parts with WD40, especially the sliding brackets ⑥.
- Check all electrical connections to the control box ⑦.
- Put the unit cover back again and tighten the retaining screws.
- Reconnect mains.

Trouble-shooting


Symptom	Cause	Action
The nursing bench does not stay in upright position.	The gas cylinders is defective.	Contact Pressalit A/S or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The nursing bench runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.


Technical data

Max load	150 kg /330 lbs
Materials	Powder laquered stainless steel Plastics: PVC (canvas) Polystyrene (cover)
Weight	R8400: 41.56 kg, 91.62 lbs R8401: 43.02 kg, 94.84 lbs R8402: 44.46 kg, 98.01 lbs R8403: 45.92 kg, 101.23 lbs
IP class	The product has an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.
Power specifications	
Max consumption	70 W
Voltage	Mains connection: 230V/120V/100V
Frequency	50/60 Hz
Current	0,3 A


Dimensions (mm)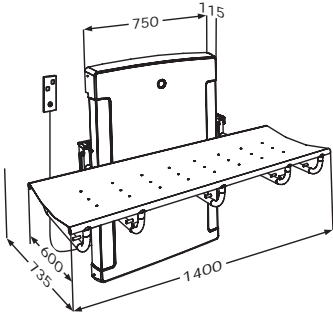


R8400

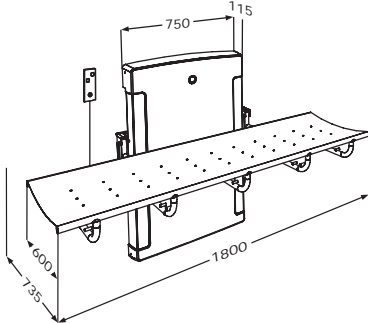R8401

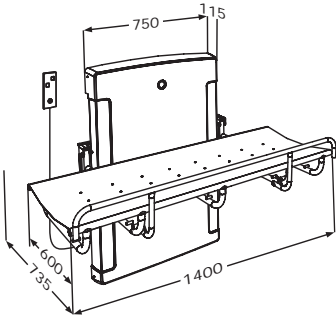
R8402

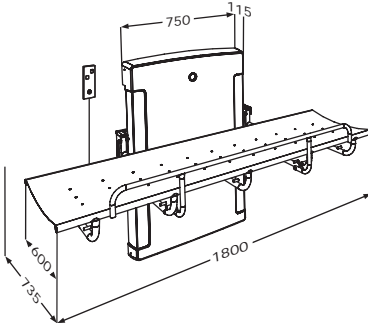
R8403

en


Dimensions (inches)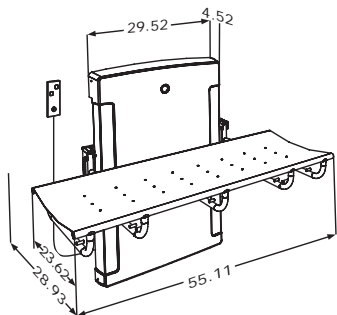


R8400

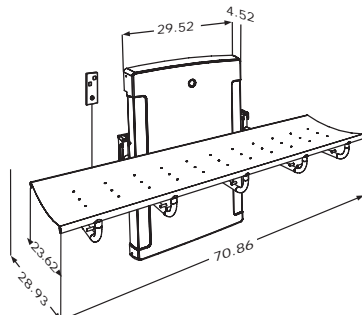R8401

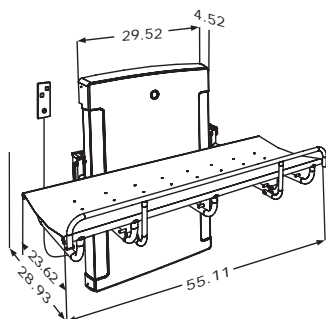R8402

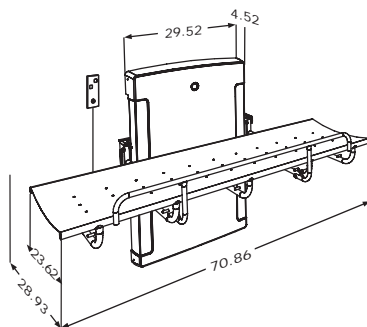R8403

en

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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OpeMed (Europe) Ltd